

Multifunction Edger ME-1500



Dramatic fit and finish

NIDEK's flagship model, the ME-1500, boasts comprehensive technology to deliver expert lens edging. True to its name of "multifunction edger", the expanded features such as drilling, high base curve lens processing, and design functions allow processing a wide array of frames.

NIDEK's exclusive processing mechanism results in a high percentage of one-cut fit accuracy and delivers edging like a master craftsman's work. With an ergonomic and compact design, plus a colorful LCD touch screen, it gives the operator a masterful command of all the functions.

The ME-1500 pairs seamlessly with the NIDEK's peripheral products like blockers and tracers, ensuring stable data management and workflow. Our portfolio of products provides an ideal system to suit any requirement. The ME-1500 satisfies your demands for meticulously finished eyewear.










Step beveling / partial step processing

With step beveling, Rx lenses can be easily inserted into sunglass frames, which are traditionally difficult to fit. The partial step processing grinds Rx lenses for specialty-type sport frames.

Maximum lens size: ø67 mm (type PLB-8S), ø72 mm (type PLB-2R8S)

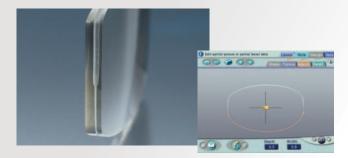

Partial grooving/beveling

Multiple grinding conditions can be applied to a lens, such as partial grooving and beveling.

Changing the groove width and depth at different locations is possible, along with partial bevel.

Design cut

Bespoke lens shapes can be designed creatively by utilizing the drill bit as a milling tool.



Facet

The ME-1500 creates a highly fashionable facet on a lens edge by setting the position and width.

The finished design of front and rear facet can be viewed on the screen in advance for accurate processing.





Intuitive 3D bevel simulation

Bevel is simulated in 3D, on the screen, to observe the real-time actual shape and position, from any angle.



Integrated accessory storage compartment

Accessories can be stored and easily accessed at any time.




Advanced shape editor

Customized shapes are easily created with the advanced shape editor. A special "fixed area" function allows the operator to change a particular portion of the lens while leaving other peripheral areas unchanged.



Data management

The ME-1500 has vast number of memory storage for basic job data, as well as custom design data files.

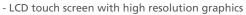

- Jog dial for smooth data entry
- Information bar indicates next operation
- Processing time indicator
- Audio announcement for processing time and completion
- Automatic processing chamber door
- Mini and nano cup (optional) for minimal B lens processing

System configurations


► Combination with ICE-1500



► Combination with LT-980 and ICE-1



► High-volume processing system*

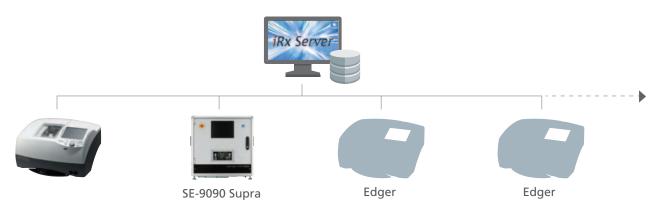

*The ME-1500 is compatible with the VCA protocols.

Wheel configuration	PLB-8S	PLB-2R8S
Plastic bevel	•	•
Plastic bevel polish	•	•
Plastic flat	•	•
Plastic flat polish	•	•
Glass bevel		•
Glass flat		•
Plastic high base curve bevel*	•	•
Step bevel / Partial step	•	•

 $[\]ensuremath{^{\star}}$ Safety beveling is not possible for high base curve bevel.

: Available

Minimum grinding size	Pliable cup (standard) W x H mm	Mini cup (optional) W x H mm	Nano cup (optional) W x H mm
Flat edging	ø32.0 x 19.5	ø22.0 x 17.4	ø20.0 x 15.5
Bevel edging	ø33.0 x 21.0	ø23.0 x 18.4	ø21.0 x 16.5
High base curve beveling	ø39.0 x 26.0	ø29.0 x 24.4	ø27.0 x 22.5
Safety beveling (flat)	ø34.5 x 21.5	ø24.5 x 19.9	ø23.0 x 18.5
Safety beveling (bevel)	ø35.5 x 22.5	ø25.5 x 20.9	ø24.0 x 19.5
Grooving	ø32.0 x 19.5	ø22.0 x 17.4	ø20.0 x 15.5

MF-1500 Specifications

Grinding system	Patternless
Mode	Beveling (automatic, guided, frame curve)
	Partial beveling (automatic, guided, frame curve)
	Flat edging
	Polishing
	Chamfering (with, without polish)
	Special safety beveling
	Facet
	Grooving (automatic, guided)
	Partial grooving
	Drilling
	Design cut
	High base curve beveling
	Step beveling
	Partial step processing
	Mini beveling (0.4 to 0.7 mm) (0.1 mm increments)
	Custom beveling
	Frame changing
	Soft processing
Setting range	p
FPD	30.00 to 99.50 mm (0.01 mm increments)
PD	30.00 to 99.50 mm (0.01 mm increments)
1/2PD	15.00 to 49.75 mm (0.01 mm increments)
Optical center height	0 to 15.0 mm (0.1 mm increments)
Size adjustment	0 to ±9.95 mm (0.01 mm increments)
Minimum grinding size	0 to 13.33 mm (0.01 mm mclements)
Flat edging	ø32.0 x 19.5 mm / with nano cup (optional) ø20.0 x 15.5 mm
Bevel edging	ø33.0 x 21.0 mm / with nano cup (optional) ø21.0 x 16.5 mm
3 3	ø34.5 x 21.5 mm / with nano cup (optional) ø23.0 x 18.5 mm
Safety beveling (flat)	
Safety beveling (bevel)	ø35.5 x 22.5 mm / with nano cup (optional) ø24.0 x 19.5 mm
High base curve beveling	
Grooving Drilling	ø32.0 x 19.5 mm / with nano cup (optional) ø20.0 x 15.5 mm
Hole diameter	ø0.80 to 10.00 mm (0.01 mm increments)
	6.0 mm or less
Hole depth	ø33.0 to 70.0 mm from lens rotation axis
Range for hole milling	
Direction for hole milling	
Slotted hole width	ø0.80 to 10.00 mm (0.01 mm increments)
Slotted hole depth	6.0 mm or less
Slotted hole length	20.8 mm or less
Wheel configuration	Type PLB-8S, PLB-2R8S
Water supply system	Pump circulation or direct connection to tap water
Interface	RS-232C - 3 ports
	1 port for connection with a computer or blocker
	1 port for connection with a barcode scanner
	1 port for connection with a frame tracer
	USB - 1 port
	LAN - 1 port
Power supply	100 to 120 V AC / 200 to 240 V AC, 50/60 Hz
Power consumption	1.5 kVA
Dimensions/mass	600 (W) x 496 (D) x 355 (H) mm / 52 kg
	23.6 (W) x 19.5 (D) x 14.0 (H)" / 115 lbs.
Standard accessories	Pliable cup, Pliable cup for high base curve lenses, Double-coated adhesive pad,
	Stylus pen, Pliable cup remover, Compound kit for polishing wheel, Adapter set
	Flat lens, Dressing stick for finishing wheel,
	Dressing stick for glass roughing wheel (type PLB-2R8S),
	Hexagonal screwdriver, Hexagonal wrench, RMU/LMU calibration jig, Wrench,
	Tray, Drill bit, Power cord, Ferrite core
Optional accessories	Barcode scanner, Internal barcode scanner, Circular pump tank, Mini cup set, Nano cup kit, USB flash drive, Drill bit (ø1.0, 1.2, 1.6)

Specifications and design are subject to change without notice.



HEAD OFFICE (International Div.) 34-14 Maehama, Hiroishi-cho, Gamagori, Aichi 443-0038, JAPAN TEL: +81-533-67-8895 URL: www.nidek.com [Manufacturer]

TOKYO OFFICE (International Div.) 3F Sumitomo Fudosan Hongo

Bldg., 3-22-5 Hongo, Bunkyo-ku, Tokyo 113-0033, JAPAN TEL: +81-3-5844-2641 URL: www.nidek.com

NIDEK INC. 2040 Corporate Court, San Jose, CA 95131, U.S.A.

TEL: +1-408-468-6400 +1-800-223-9044 (US Only) URL: usa.nidek.com

NIDEK S.A. Ecoparc, 9 rue Benjamin Franklin, 94370 Sucy En Brie, FRANCE TEL: +33-1-49 80 97 97

URL: www.nidek.fr

Via dell'Artigianato, 6/A, 35020 Albignasego (Padova), ITALY

NIDEK TECHNOLOGIES S.R.L.

TEL: +39 049 8629200/8626399 URL: www.nidektechnologies.it NIDEK (SHANGHAI) CO., LTD. Rm3205,Shanghai Multi Media Park, No.1027 Chang Ning Rd, Chang Ning District, Shanghai, CHINA 200050 TEL: +86 021-5212-7942

URL: www.nidek-china.cn

NIDEK SINGAPORE PTE. LTD. 51 Changi Business Park Central 2, #06-14, The Signature 486066, SINGAPORE

TEL: +65 6588 0389

URL: www.nidek.sg